


Syndeo System Cleaning Protocol




Mandatory End Treatment Rinse

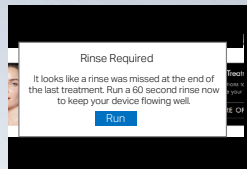

End of treatment
Rinse Prompt



Put the cap on and turn flow
to fully open +

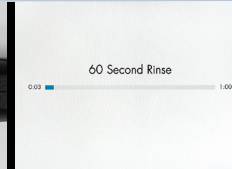


Perform 60 second
rinse



Empty, rinse and dry
the gunkie jar


End Of **Every Day**

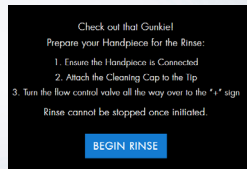

Rinse prompt



Put the cap on and turn
flow to fully open +

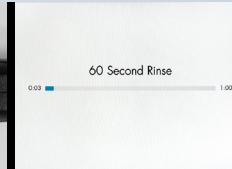


Perform rinse



Empty, rinse and dry

Handpiece Cleaning – Weekly

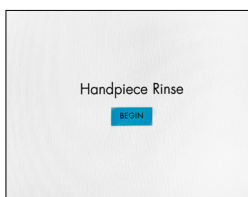

On the menu screen,
select Handpiece Rinse
from the System Rinses.



Open control valve
to the +.



Detach the handpiece
from all tubing by gently
twisting the knobs
counter-clockwise.




Remove the cleaning cap, then
insert the cleaning brush into
the vacuum (smaller) opening
on the face of the handpiece.
Push the brush in and out fully
while rotating three times.
Repeat in blue connector.



Blow air into the blue return line
two times.




Clean the external handpiece and aluminum
head with a germicidal wipe.

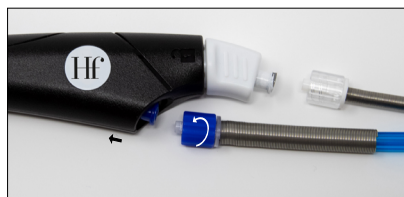

Re-attach the tubing.




Put on the cleaning cap.

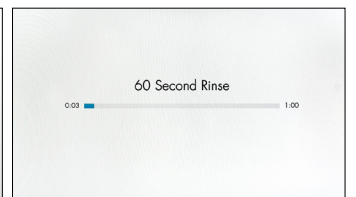

Perform a 60-second
system rinse.

Mandatory Manifold Cleaning

Every 7 Days from First Treatment Run Since Last Manifold Clean.

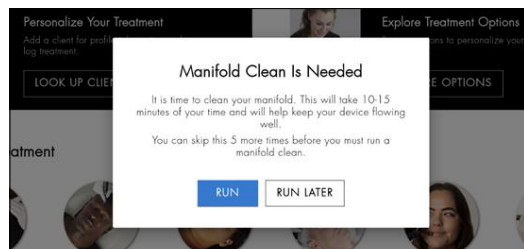


On the screen, select Menu. Then, select Manifold Rinse from the System Rinses.




Remove serums from the manifold.



Place a towel under the serum straws to capture liquid.

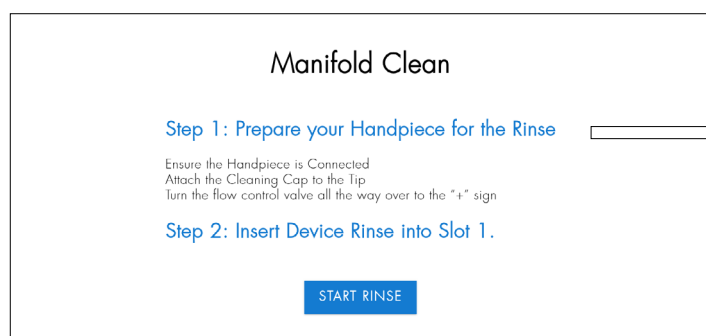

Ensure the Handpiece is Connected
Attached the Cleaning Cap to the Tip
Turn the flow control valve all the way over to "+" sign

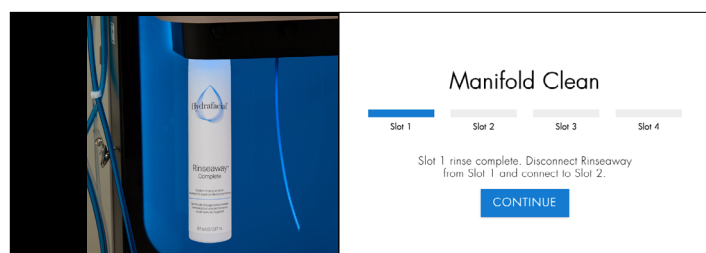


Insert Device Rinse bottle into Slot 1. Select Start Rinse.



When rinse is complete, disconnect Device Rinse bottle from Slot 1 and connect to Slot 2. Select Continue.



When rinse is complete, disconnect Device Rinse bottle from Slot 2 and connect to Slot 3. Select Continue.

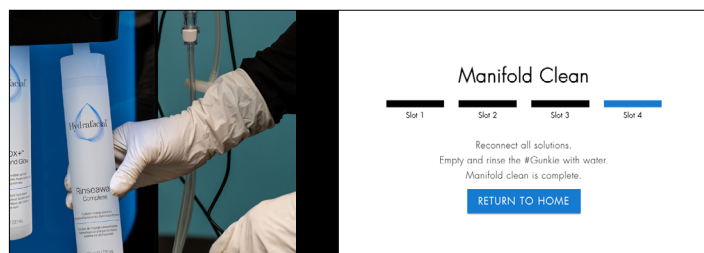

When rinse is complete, disconnect Device Rinse bottle from Slot 3 and connect to Slot 4. Select Continue.


Make sure to press 'Return Home' after completing Slot 4.



Replace the remaining serum bottles and clean the Gunkie Jar.


Emergency Rinse Skip Options

Skipping Rinses are sometimes necessary in emergency situations, however, your system will operate at its best when all Rinses are performed.

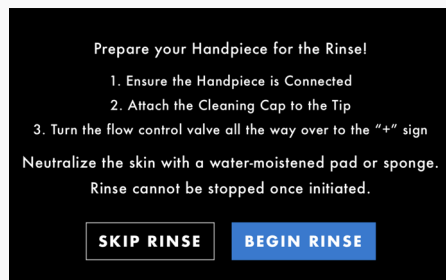


Mid-treatment Rinse

May be skipped.

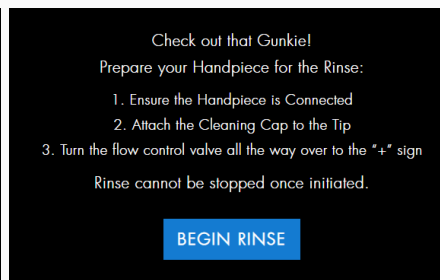

End Treatment Rinse

Can be skipped up to 5 times in 30 days.

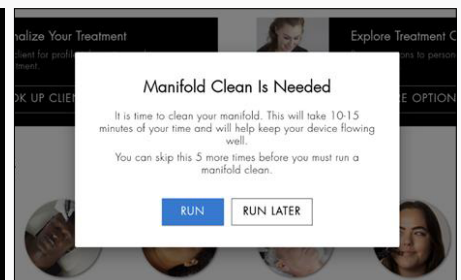


Manifold Clean

Can be skipped up to 5 times before it is required.

Note: Rinses are critical to help your system stay functioning at its best. Once you have reached the maximum number of skips allowed your system will be not be able to perform treatments.

LED Lights

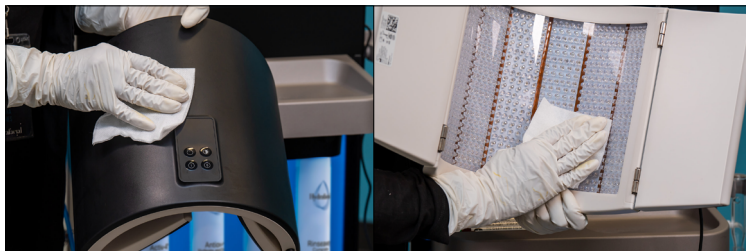

To Clean: Unplug the device and wipe softly with a slightly damp soft towel, then dry with a soft towel.

To Disinfect: Unplug the device from the power source. Wipe the surface of the device with a Caviwipe disinfectant towelette for at least 30 seconds.


Approved disinfectants are: CaviWipes Towelettes.

Lymphatic Therapy



Clean the lymphatic handpieces with a germicidal solution on any area that makes contact with the skin.

System Touchscreen

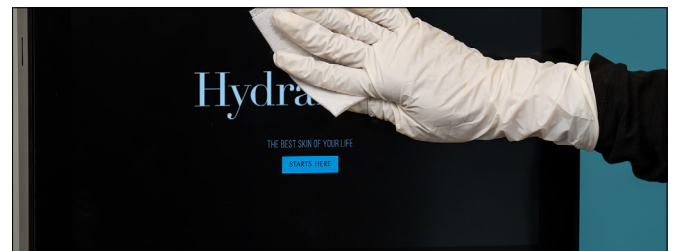

Use a mild glass cleaner and a microfiber cloth or towel to clean the system touch screen. Spray on the cloth, not directly on the screen.

Wipe down the system and screen with a microfiber cloth regularly to prevent buildup of dust, debris, and dirt.